

Description

Remescar assists in the healing process of both new and old scars resulting from surgery, burns, bug bites, acne, stretch marks, etc., by releasing a thin and discrete silicone-based film to provide a suitable healing micro-environment for the scar while maintaining the skin's natural moisture balance.

Intended use

Remescar is designed for the management of old and new scars.

It can also be used as a preventive therapy on closed wounds to reduce scarring.

The following types of scars can be treated:

- Keloid scars
- Hypertrophic scars
- Burn scars
- Trauma scars
- Stretch marks
- Post operative scars
- Body contouring scars
- Body piercing scars
- Post laser peel redness
- Acne scars
- Bug bite scars

Instructions for use

The scarred area should be closed, clean and dry. At first time use, open the stick and remove the protective cap. Raise the stick not more than 1/8 of an inch for application. It must be raised enough so the plastic will not rub against your skin. Remescar should be applied gently on the affected area and around the border of the scar. Do not rub too hard as this will not increase the effectiveness.

Screw the stick down after use and close.

Apply Remescar twice a day for 2 to 3 months.

Precautions

- Not to be applied on third degree burns.
- For external use only.
- Not to be used on dermatological conditions that disrupt the integrity of the skin.
- Do not use on open or infected wounds.

- Do not use on skin currently affected by acne.
- Only use on normal healthy, non-infected skin.
- Use of the product is inadvisable if you have allergic reactions to one of the ingredients.
- Should not be placed in contact with mucous membranes or the eyes.
- Should not be applied over other skin treatments without the advice of your physician.
- May stain clothing if not completely dry.
- If skin irritation occurs, discontinue use and consult your physician if the irritation persists.
- Keep out of the reach of children. Store below 30°C.
- Do not use after the expiry date.
- Once opened, it's advised to use the product not longer than 6 months.
- Remescar is not sterile and must not be used as a wound dressing.
- Not recommended for children under the age of 6 without supervision. For children less than 1 year consult a professional before using the product.

Notice: any serious incident that has occurred in relation to this device should be reported to the manufacturer and the competent authority of the Member State in which you are established.

Sylphar NV – Xavier De Cocklaan 24
9831 Deurle - Belgium
www.remescar.com

We'd love to hear from you and get any feedback you have on our product:



Scan me!

Symbols

- Manufacturer
- Medical device
- Batch number
- Expiry date
- Keep away from direct sunlight
- Read the instructions of use
- Keep dry
- Do not use if package is damaged
- Single patient, multiple use

CE
0197

